

**GETINGE 46-SERIES  
WASHER-DISINFECTORS**



Always with you

**How many reusables do you have to process?**  
**What kinds?** Getinge 46-series includes machines precisely matched to your needs.

**Additional options**

- Flow monitoring – to measure the flow from the detergent pumps and stop the process if the set quantity has not been dispensed.
- Printer – free-standing or mounted on the machine.
- Extra dosing pumps – two pumps are standard, but Getinge 46-5 and 46-4 can be fitted with two extra pumps, while Getinge 46-2 takes only one optional extra pump.
- Cover plates – for building into the barrier.
- Pressure monitoring – checks the water pressure.
- Drain cooling – cools the drained water.
- Purified water valve – for final rinse water
- Purified water pump – for low-pressure final rinse water
- Filter monitoring – checks the sterile filter
- Conductivity control – to verify that no detergent residues are left in the final rinse water, and thus not in the disinfected items either.
- Acoustic signal – in the event of an alarm.



# NEITHER MORE NOR LESS

The new Getinge 46-series washer-disinfectors are precisely matched to every need in the healthcare sector. Choose between two, four and five levels, depending on the type and quantity of items you need to process.

**The Getinge 46-series** includes machines for wards, outpatient clinics, surgical departments and central sterile supply departments. These washer-disinfectors process all kinds of reusables – from complicated instruments to ward utensils. All kinds of tubular instruments can be cleaned, disinfected and dried.

## A cost-effective choice

A washer-disinfector with suitable accessories to meet your demands will be cost-effective from Day One. Getinge 46-series offers the right model for every department.

## High washing capacity

Getinge 46-series washer-disinfectors combine large chambers with small external dimensions. The Getinge 46-5 model has 5-level instrument wash carts, Getinge 46-4 has 4-level wash carts.

Each shelf and spray arm is removable, and the spray-arm fixtures can be used for other accessories, e.g. for cleaning of complicated items such as tubular instruments.



**Getinge 46-2** – for wards and outpatient clinics, items on two levels.



**Getinge 46-4** – for surgical departments and smaller central sterile supply departments, items on four levels.



**Getinge 46-5** – for larger surgical instrument sets and central sterile supply departments, items on five levels.

Washer-disinfector Getinge 46-series	WASHER-DISINFECTOR MODEL		
	46-2	46-4	46-5
Wash levels	2	4	5
Chamber height	590 mm	590 mm	660 mm
Chamber width	550 mm	550 mm	550 mm
Chamber depth	620 mm	620 mm	620 mm
DIN 1/1 trays per load*	4	8	10
SPRI II trays per load**	4	8	10

\* Tray size: 485x250x50 mm

\*\* Tray size: 340x250x70 mm

GETINGE

46-4




ROT. OP-SHIFT-D  
TEMP. INDEP. 86.1.5%



## EXPERIENCE THE LATEST TECHNOLOGY

The unique Getinge 46-series is the result of input from users and the carefully thought-out ideas of our industrial designers. Hygienic and ergonomical solutions are combined with the latest technology for maximum user security – including built-in alarms.

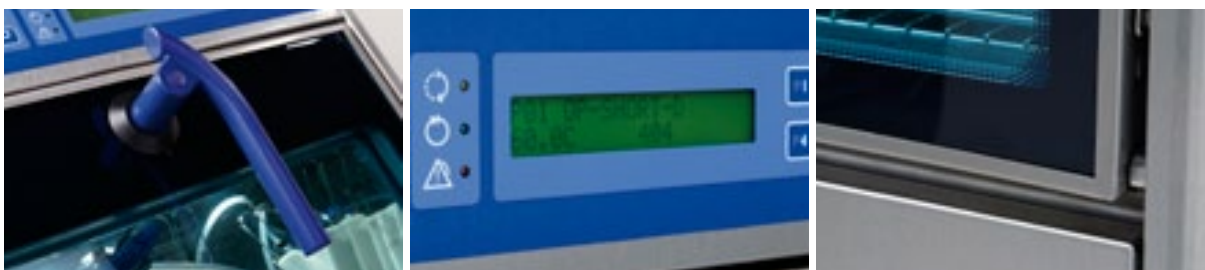

### Meet's all standards

The Getinge 46-series offers superior cleaning and disinfection. These washer-disinfectors meet all relevant standards of today including the ISO EN 15883.

### Design with function

The design of the new Getinge 46-series is based on input from users and service technicians regarding good function:

- **GLASS DOOR** – provides full visual inspection of the process, standard on all machines in the Getinge 46-series.
- **ERGONOMIC HANDLE** – makes it very easy to open and close the door.
- **SMOOTH SURFACES** – easy to wipe, no unhygienic dirt traps.
- **FEW PANELS** – easy to remove, convenient to service from the front and easy to clean.

### Control system

The processes in Getinge 46-series washer-disinfectors are safeguarded by the PACS 350 control system. The system has many built-in safety functions and monitoring options, which assures satisfactory results every time.

### Effective and economical drying

The washer-disinfectors in the Getinge 46-series dry items effectively on the inside and outside. Drying time and temperature can be preset for each program via the control panel.

### Easy to use

Select the program and press “start” – your Getinge 46 will deal with everything else! You can then track in plain language on the display everything that happens in every stage of the washing and disinfecting processes.

### Connect to T-DOC

The Getinge 46-series is prepared for connection to Getinge T-DOC – a software-based quality assurance system that traces and documents your production. It's more convenient and reliable than paper-based systems, and is so efficient that you'll quickly recoup your investment.

T-DOC stores all process information from your machines in one and the same place. As a result, you can study a process at any time on the basis of a graph or the numerical log values. Quickly, securely and simply!

# GETINGE 46-2

## FOR WARDS AND OUTPATIENT CLINICS

The Getinge 46-2 washer-disinfector has been specially developed to meet the needs of wards. It cleans and disinfects recirculated items and instruments – even tubular ones – on two levels. Specially adapted accessories (see below) optimize the capacity of the machine.



**Broad range of accessories** There are many effective and time-saving accessories for the Getinge 46-series. Some are shown in this brochure, but for the complete selection, please order our "Getinge 46-series accessories catalogue". Note that the accessories in the former Getinge 4656 also fit the new Getinge 46-series.



Bowls 2 levels



Baby feeding bottles



Lower wash cart with L-shaped injector bar



# GETINGE 46-4 FOR SURGICAL DEPARTMENTS

The Getinge 46-4 has been specially developed to match the needs of surgical departments and smaller central sterile supply departments. It cleans and disinfects all kinds of items on four levels.

Each wash level, consisting of a removable shelf and spray arm, can be adapted to each individual need. Tubular instruments can be conveniently placed in an MIS wash cart, which can take as many as 80 connections. The AN wash cart has capacity for 4–6 AN sets with a maximum of 12 tubes.



**Broad range of accessories** There are many effective and time-saving accessories for the Getinge 46-series. Some are shown in this brochure, but for the complete selection, please order our "Getinge 46-series accessories catalogue". Note that the accessories in the former Getinge 4656 also fit the new Getinge 46-series.



4-level instrument wash cart



Universal lower wash cart and basic upper wash cart



AN wash cart



# GETINGE 46-5

## FOR SURGICAL DEPARTMENTS AND CENTRAL STERILE SUPPLY DEPARTMENTS

The Getinge 46-5 has been specially developed to match the needs of larger surgical instrument sets and in central sterile supply departments. It cleans and disinfects all kinds of items on five levels.

Each wash level, consisting of removable shelf and spray arm, can be adapted to the individual need. Tubular instruments can be advantageously cleaned in the MIS wash cart with capacity for as many as 80 connections. The AN-wash cart has capacity for 4–6 AN sets with up to 12 tubes.

### Unique drying unit

- Effective, fast and energy efficient
- Extremely fast heating up of the heating elements
- Energy-efficient heat exchanger

### Goods on five levels

### High standard of hygiene

Smooth chamber without welded joints or pockets where bacteria can grow.


### Large chamber

H660xW550xD620 mm.

### Small external dimensions

Width only 650 mm – takes up little space in the disinfection room.

Note! Getinge 46-5 is the same size as the older Getinge 4656, so replacement is easy.

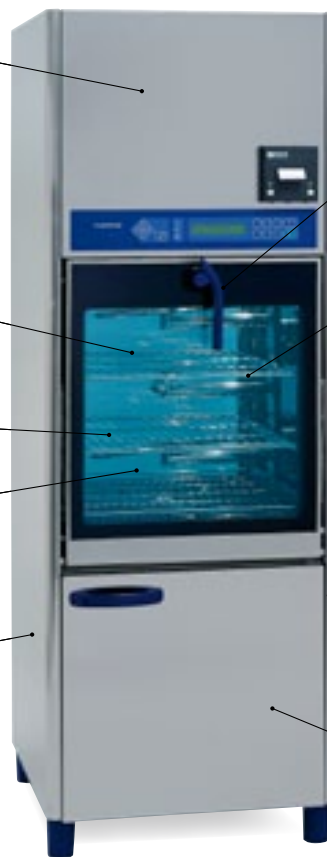

### Ergonomic handle

### Glass door

Makes it easy to overview the process

### Easy to clean

No external screws that can gather dirt.

### Convenient to service

Few screws need to be loosened.

### Built-in storage

For different types of detergents.



**Broad range of accessories** There are many effective and time-saving accessories for the Getinge 46-series. Some are shown in this brochure, but for the complete selection, please order our "Getinge 46-series accessories catalogue". Note that the accessories in the former Getinge 4656 also fit the new Getinge 46-series.



5-level instrument wash cart



Container wash cart



MIS wash cart





#### COMPLETE SOLUTIONS FOR INFECTION CONTROL

Getinge is the world's leading provider of solutions for effective cleaning, disinfection and sterilization in the healthcare and life science sectors. We are dedicated to helping our customers provide better care at a lower cost. We do this by offering well thought through and customized solutions. This means that we are with our customers all the way from architectural planning and education to traceability and support – with complete solutions, long-term commitment and global presence. Getinge – Always with you.

#### Getinge Infection Control AB

P O Box 69  
SE-310 44 Getinge  
SWEDEN  
Phone: +46 35 15 55 00  
Fax: +46 35 18 14 50

**GETINGE**  
GETINGE GROUP

[www.getinge.com](http://www.getinge.com)

GETINGE GROUP is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of ArjoHuntleigh, GETINGE and MAQUET. **ArjoHuntleigh** focuses on patient mobility and wound management solutions. **GETINGE** provides solutions for infection control within healthcare and contamination prevention within life sciences. **MAQUET** specializes in solutions, therapies and products for surgical interventions and intensive care.